




*COSMETRICS*TM


Installation Guide

COSMETRICS Software Installation Instructions

1. Place the COSMETRICS software CD into your computers CD-ROM drive.
2. The CD will run automatically and after a few moments the Installation wizard will appear on the screen. Follow the wizard by **Clicking** on the **Next** button.

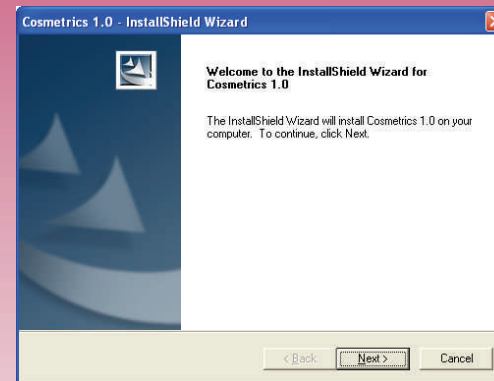


- Note:** If the CD does not start Automatically **Double Click** on the **CD Icon**. This can be found by **Clicking** on **“My Computer”**.
3. Accept the terms of the license agreement by **Clicking** on the radio button and **Click** on the **Next** button.

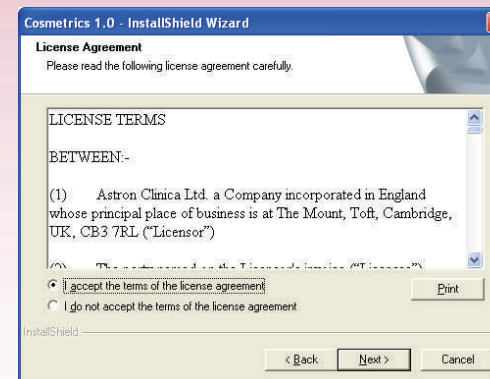


COSMETRICS Software Installation Instructions

4. Select **Full Install** (recommended unless you are an experienced user) from the options shown. Then **Click** on the **Next** button.


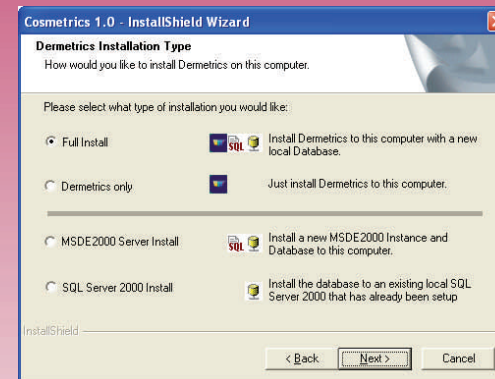


5. Please read the Windows Security Information and then **Click** on the **Next** button.

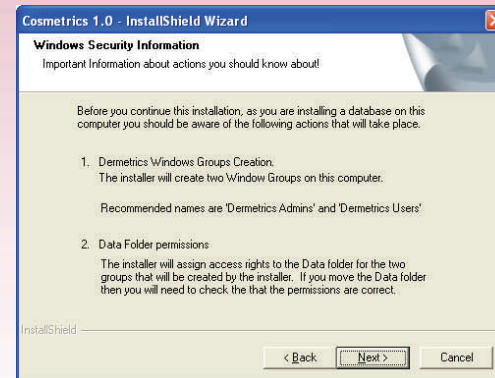


COSMETRICS Software Installation Instructions

6. Accept the default location (recommended unless you are an experienced user) for the set up location folder and **Click** on the **Next** button.
7. Accept the default handset set up unless you have been instructed to choose differently by Astron Clinica's Technical support team. Continue by **Clicking** on the **Next** button.

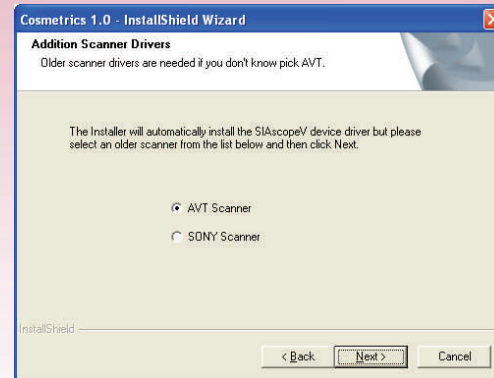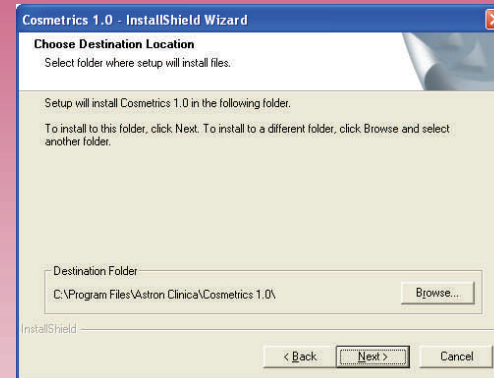


COSMETRICS Software Installation Instructions

8. Accept the default location (recommended unless you are an experienced user) for the database folder and **Click** on the **Next** button.

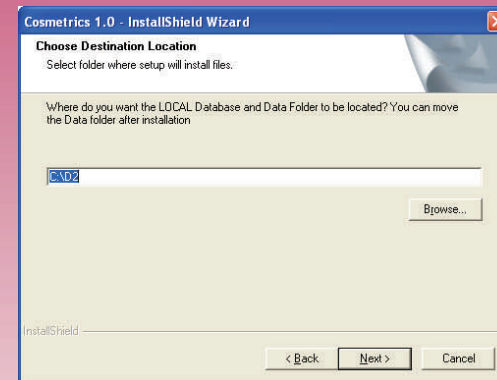

9. Accept the default names for the server and database names (recommended unless you are an experienced user) and **Click** on the **Next** button.

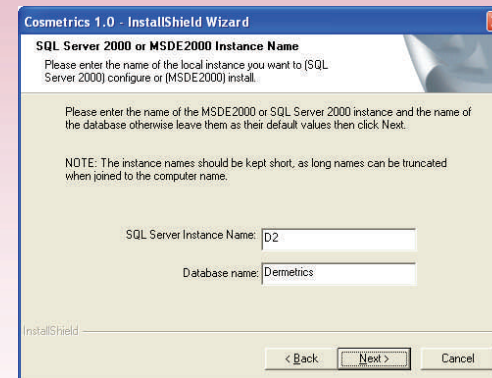

COSMETRICS Software Installation Instructions

10. Accept the default names of the access groups (recommended unless you are an experienced user) that are going to use the software and **Click** on the **Next** button.


The screenshot shows a Windows dialog box titled "Cosmetics 1.0 - InstallShield Wizard". The main heading is "Windows Security Group Names". Below the heading, it says "Please enter the Windows Group names, they will be created and given access to the Database and Data folder." There is a paragraph of text: "Please enter the names of the two Local Windows Groups that are going to be used to access the data used in Demetrics, then click Next." Another paragraph explains: "Users that are added to the Demetrics Admins Group will have additional rights to perform upgrades and other administrative tasks whereas Users that are added to the Demetrics Users Group will only be able to use the Demetrics software." There are two text input fields: "Demetrics Admin Group Name:" with the value "Demetrics Admins" and "Demetrics User Group Name:" with the value "Demetrics Users". At the bottom, there are three buttons: "< Back", "Next >", and "Cancel".

11. All of the installation options have been set. To complete the software installation **Click** on the **Install** button.


The screenshot shows a Windows dialog box titled "Cosmetics 1.0 - InstallShield Wizard". The main heading is "Ready to Install the Program". Below the heading, it says "The wizard is ready to begin installation." There is a paragraph of text: "Click Install to begin the installation." Another paragraph explains: "If you want to review or change any of your installation settings, click Back. Click Cancel to exit the wizard." At the bottom, there are three buttons: "< Back", "Install", and "Cancel".

COSMETRICS Software Installation Instructions

12. The installation will continue and the progress bar will be updated during the process.

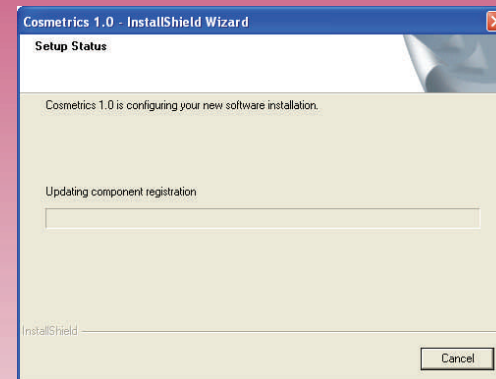

13. When the installation is complete **Click** on the **Finish** button. It is advised to restart your computer to complete the installation.

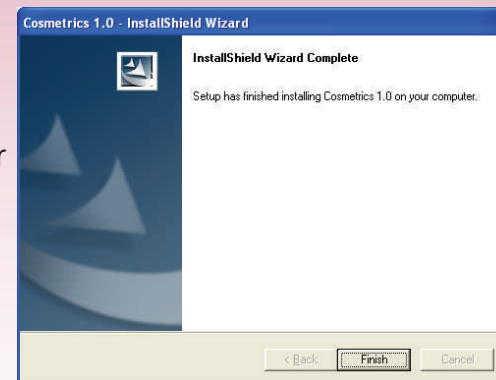

Software Installation Instructions

The COSMETRICS software installation is complete. The USB software license key can now be installed.


Place the USB software license key into a spare USB 2.0 port on your computer. The key will automatically be recognised by your operating system and a message will appear in the bottom right hand corner of your desktop confirming a successful installation.



The final part of the installation is to connect the SIAscope IV . Please follow the instructions in this guide.

COSMETRICS SIAscope IV Installation Instructions

1. Plug the USB 2.0 cable into the SIAscope IV and into an available USB 2.0 port on your computer.

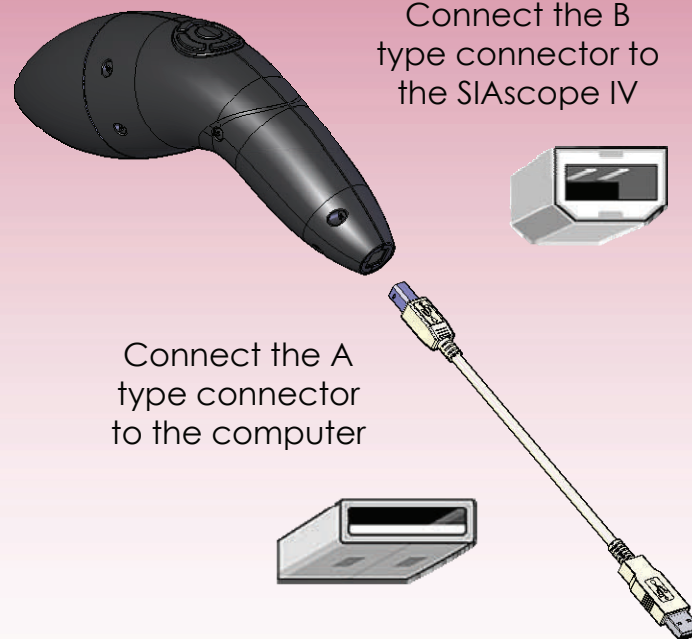


COSMETRICS SIAscope IV Installation Instructions

2. The first time a SIAscope IV is used a hardware wizard will be displayed. In the first window select “**No Not At This Time**” and **Click** on **Next**.




3. Now select **Install From a List or Specific location (Advanced)** and **Click** on **Next**.

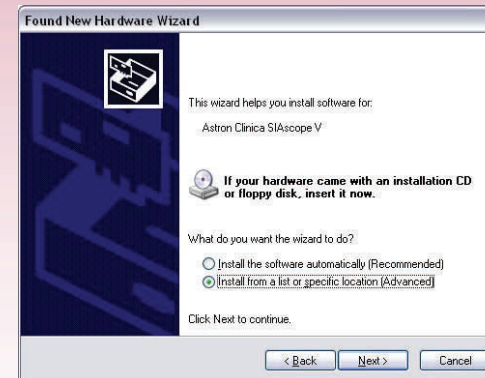

COSMETRICS SIAscope IV Installation Instructions


4. Select “**Don’t Search. I will choose the driver to install**” and **Click** on **Next**.

5. The text Astron Clinica SIAscope IV will be displayed in the window. Now **Click** on **Next**.

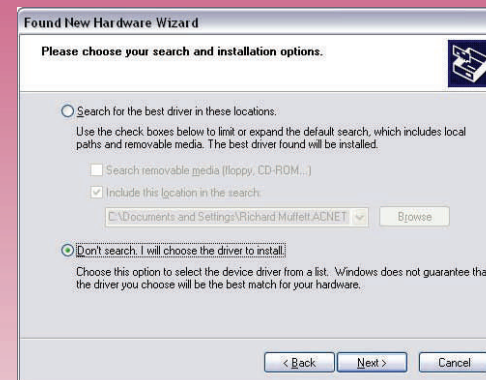

COSMETRICS SIAscope IV Installation Instructions

6. The wizard will complete the installation and when displayed **Click** on the **Finish** button.

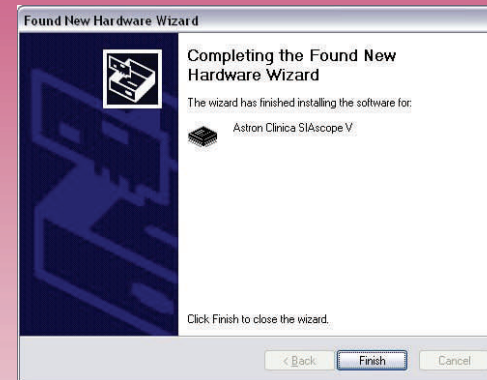

7. If a problem occurs with the installation of the SIAscope IV and the message reads device can not start (code 10) in the image shown in step 5 then try the following to correct the problem:
 - a. Repeat the procedure but at step 5 **Click** on the **Have Disk** button.
 - b. Navigate to **C:\Program Files\Astron Clinica\Cosmetics 1.0\Drivers**
 - c. Select the file **SIAscopeV** and then **Click** on **OK**.
 - d. When you see the text **Astron Clinica SIAscope V** again as in step 5 originally follow the instructions as before.

If the handset still fails to install correctly please contact Technical Support.



Document Revisions

Revision	Date	Description
0.1	22/09/2006	First draft composed
0.2	25/09/2006	Final version for print

The logo for COSMETRICS features the word "COSMETRICS" in a white, sans-serif font, positioned to the left of a white, semi-circular arc. A small, multi-colored triangular graphic is placed above the letter "S" in "COSMETRICS".

COSMETRICS Acknowledgements & Notices

SIA, SIAscanner, SIAscope, SIAscan, SIAscopy, COSMETRICS, Astron Clinica, SIAscopeV, DERMETRICS, MoleMate, MediSpa and Powered By SIAscopy are registered trademarks of Astron Clinica Ltd.

Microsoft, Windows are registered trademarks of Microsoft Corporation.

The information contained in this manual is subject to change without notice.

Astron Clinica Ltd makes no warranty of any kind with regard to this material, including but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Astron Clinica Ltd shall not be liable for any errors or for incidental or consequential damages in connection with the furnishing, performance, or use of this material.

All rights reserved. Reproduction, adaptation, or translation of this document is prohibited without prior written permission of Astron Clinica Ltd, except as allowed under the copyright laws.

No part of this document may be photocopied, reproduced or translated to another language without the prior written consent of Astron Clinica Ltd.

The SIA technology is protected by patents in most major territories around the world including Australia (Patent No. 725 766), UK (Patent No. 2 334 099) and USA (Patent No. 6 324 417).

The DERMETRICS software is subject to a separate license agreement and must not be copied or distributed to third parties.

Copyright Astron Clinica Limited 2006.